



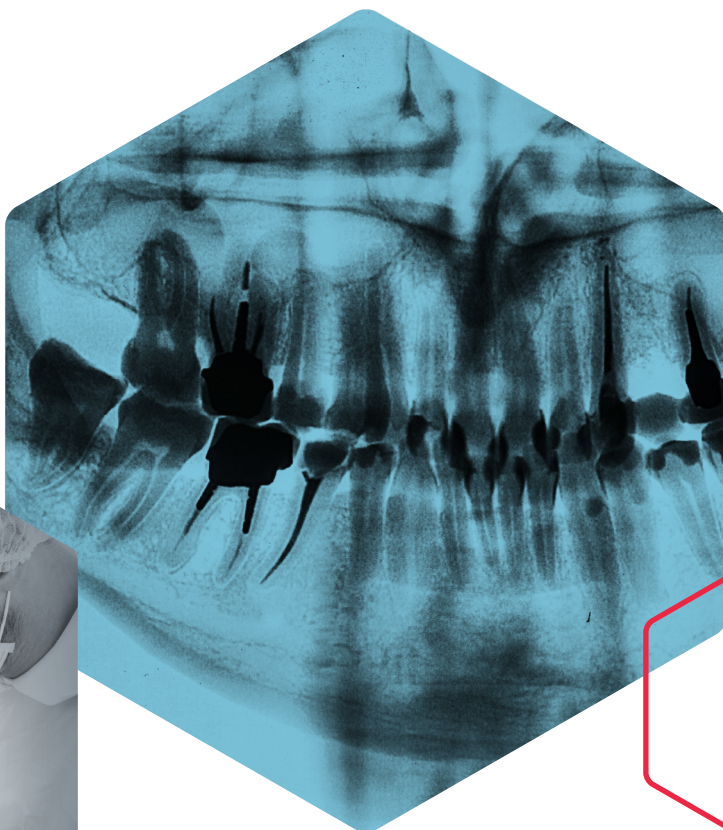

THE SAFE & FAST WAY TO NEW BONE

CERASORB® M

Resorbable, pure-phase β -tricalcium phosphate

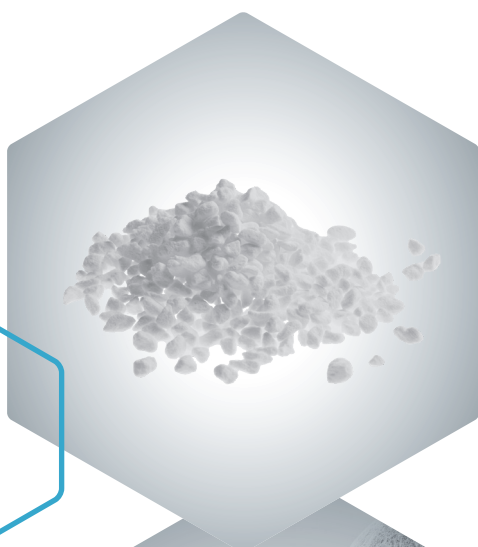

CERASORB® M is the safe and fast way to new bone. As a comprehensive product system for bone and tissue regeneration, it is fully remodeled within the natural bone regeneration process. Thanks to its superior chemical and material properties, **CERASORB® M** ensures high safety and success rates in defect restoration.

Widely studied product with
>200 yearly publications


1997

A GLOBALLY TRUSTED PRODUCT WITH

Clinically proven to be equivalent
to autologous bone.

CERASORB® M approved in:

- › Europe
- › Switzerland
- › United Kingdom
- › USA
- › Canada
- › Brasil
- › China
- › Taiwan
- › South-Korea
- › Japan
- › Saudi Arabia
- › Israel
- › South Africa

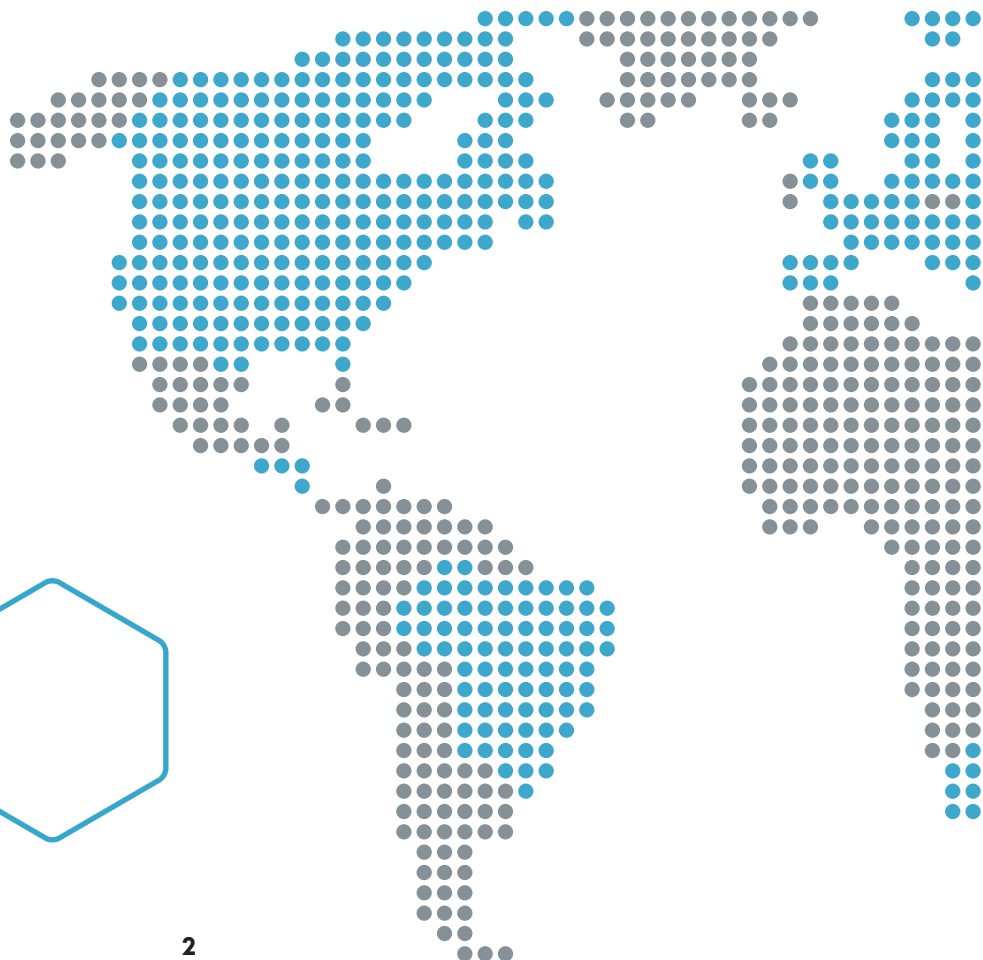

BONE REGENERATION



“Restitutio ad integrum”

Real bone regeneration – no substitution!

More than 2 MILLION Vials sold so far with ZERO reported medically related side-effects

OVER 25 YEARS

Ongoing

Transformation to bone is also confirmed in histologic examinations

CERASORB® M

Resorbable, pure-phase β -tricalcium phosphate

General description:


- › Fully biosynthetic
- › Fully resorbable
- › Biocompatible
- › Low radiodensity
- › Phase purity >99%
- › Remodels into bone within 4 – 6 months

Open multi-porous structure characterized by micro, meso and macro porous

Interconnected porosity

Polygonal shaped granules

Scaffold that mimics human bone

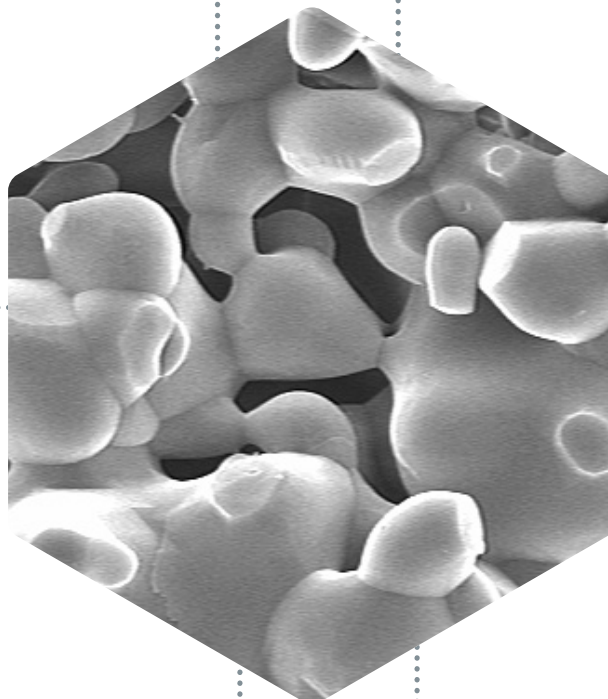

Interlocking granules

Irregular granule shape

Features & Benefits

- › 99% biosynthetic pure-phase β -TCP
- › 65% porosity to resorb uniformly and parallel
- › 100% biomimetic
- › 100% biodegradable – restores natural condition
- › Free of pathogens and pyrogens
- › No risk for allergies or infections
- › Transformation to bone is confirmed histologically
- › Reproducible human clinical results
- › No product-related medical complaints
- › Fully degradable over 4-6 months
- › Radiopaque

Indications

- › Sinus lift
- › Alveolar ridge preservation
- › Cystic defects
- › Socket and ridge preservation
- › Periodontal defects
- › Intra bony defects
- › Implant bed preparation
- › Osteotomy

Why polygonal and irregularly shaped in CERASORB® M plays an important role?



- › The polygonal granule shape enhances vascularization by providing more space between granules.
- › The granule's polygonal and irregular shape enhances intercalation and interlocking in the defect cavity avoiding micro movements that can delay the healing process.





CERASORB® M's phase purity and osteoregenerative properties

CERASORB® M sets the gold standard in biosynthetic bone grafting with >99% of its phase purity. Its highly stable and bioactive β -TCP composition closely resembles natural bone minerals, ensuring enhanced osteoconductivity, efficient bone remodeling, and predictable resorption of the material with new bone formation. As the purest β -TCP available worldwide, CERASORB® M provides a reliable and clinically proven foundation for bone regeneration.

According to Bohner et al. (2020)¹, β -tricalcium phosphate (β -TCP) exhibits potential osteoinductive properties. **CERASORB® M** is designed with an interconnected, open multi-porous structure, where its micro-porosity ($\leq 50 \mu\text{m}$) and bioactivity play a key role in being osteoregenerative.

According to studies (Yuan et al., 1998²; Yuan et al., 2001³; Wilson et al., 2006⁴; Cheng et al., 2013⁵; Kondo et al., 2006⁶), β -TCP micropores ($0.1\text{--}100 \mu\text{m}$ in diameter) significantly enhance biological response, facilitating both resorption and osteoinductive potential. Furthermore, a comprehensive review by Yuan et al. (2010)⁷ reinforces the role of biomaterials with osteoinductive potential as a viable biosynthetic alternative to autologous bone grafting, establishing β -TCP as a fundamental component in advanced regenerative dentistry and orthopedic applications.

¹ Bohner, M., Santoni, B. L. G., & Döbelin, N. (2020). β -tricalcium phosphate for bone substitution: Synthesis and properties. *Acta biomaterialia*, 113, 23–41. <https://doi.org/10.1016/j.actbio.2020.06.022>

² Yuan, H., Yang, Z., Li, Y., Zhang, X., De Bruijn, J. D., & De Groot, K. (1998). Osteoinduction by calcium phosphate biomaterials. *Journal of materials science: materials in medicine*, 9, 723–726


³ Yuan, H., De Bruijn, J. D., Li, Y., Feng, J., Yang, Z., De Groot, K., & Zhang, X. (2001). Bone formation induced by calcium phosphate ceramics in soft tissue of dogs: a comparative study between porous α -TCP and β -TCP. *Journal of materials science: materials in medicine*, 12, 7–13

⁴ Wilson, C. E., Krut, M. C., de Bruijn, J. D., van Blitterswijk, C. A., Oner, F. C., Verbout, A. J., & Dhert, W. J. (2006). A new in vivo screening model for posterior spinal bone formation: comparison of ten calcium phosphate ceramic material treatments. *Biomaterials*, 27(3), 302–314.

⁵ Cheng, L., Shi, Y., Ye, F., & Bu, H. (2013). Osteoinduction of calcium phosphate biomaterials in small animals. *Materials Science and Engineering: C*, 33(3), 1254–1260.


⁶ Kondo, N., Ogose, A., Tokunaga, K., Umezue, H., Arai, K., Kudo, N., ... & Endo, N. (2006). Osteoinduction with highly purified β -tricalcium phosphate in dog dorsal muscles and the proliferation of osteoclasts before heterotopic bone formation. *Biomaterials*, 27(25), 4419–4427.

⁷ Yuan, H., Fernandes, H., Habibovic, P., De Boer, J., Barradas, A. M., De Ruiter, A., ... & De Bruijn, J. D. (2010). Osteoinductive ceramics as a synthetic alternative to autologous bone grafting. *Proceedings of the National Academy of Sciences*, 107(31), 13614–13619.

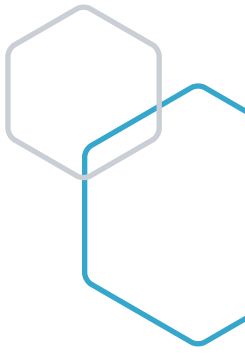



How does micro porosity stimulate osteoregenerativity?

Difference in vascularization

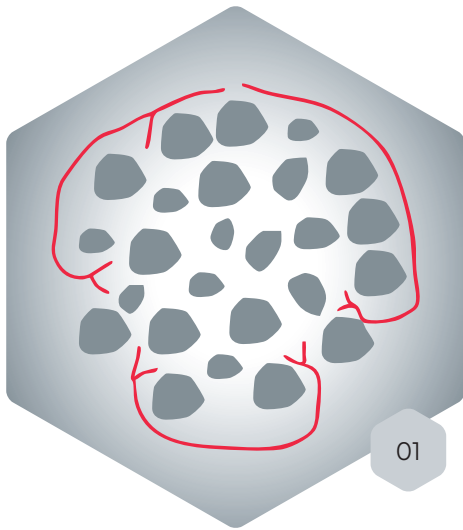

01: Products with macro and meso porous show limited vascularization

02: CERASORB® M with macro, meso, and microporosity show enhanced vascularization and bioactivity which is essential for osteoregenerativity

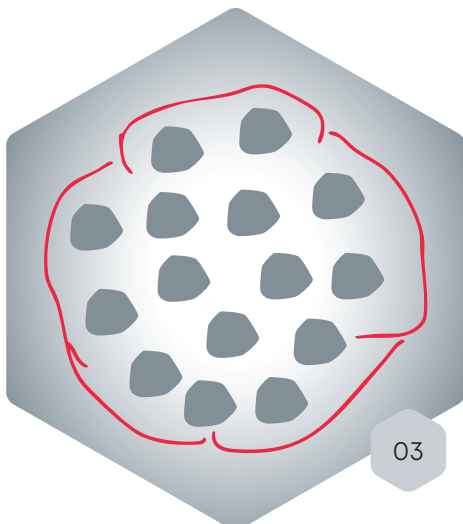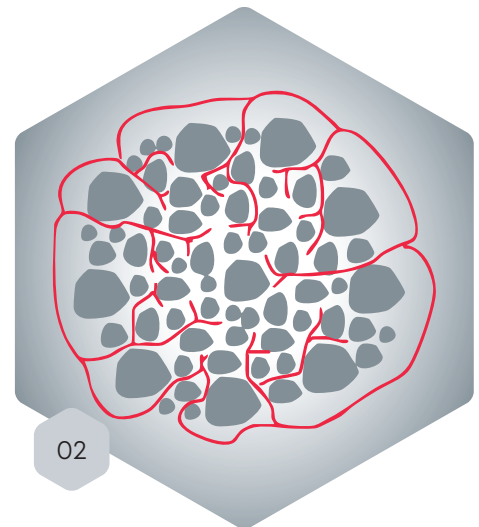

03: Products offering macro porous granules only show no sign of vascularization

Sinus floor elevation and its risk⁸

Authors: Dr. Matthias Kebernik, Prof. Dr. Dr. Frank Palm; Germany


Summary:

- › **Product used:** CERASORB® M
- › **Indications:** Sinus floor elevation




Clinical experience summary:


- › Survival rates in the region of a sinus lift range from 61.7 to 100% (91.8% on average) and do not differ significantly from the survival rates of implants placed in a non-augmented bone bed (95.7% after 9-14 years in situ).
- › Overall, implants with rough surfaces have a higher success rate than machined implants.
- › The use of autogenous bone has no advantage over a mixture of bone and bone substitute in terms of implant success.
- › In conclusion, sinus floor augmentation by sinus floor elevation is a very safe and widely used method in implant surgery.
- › It offers a high probability of success in the intersection of the dental implants in the area of the posterior maxilla and thus offers a significant expansion of prosthetic possibilities.

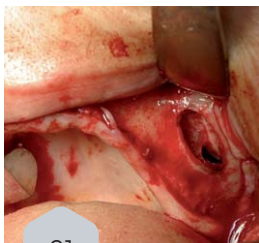


01

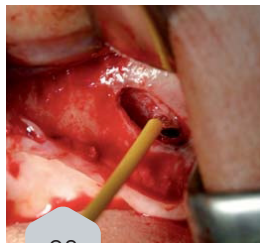


02

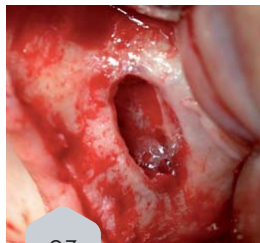


03

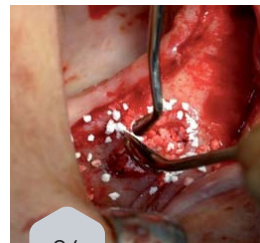

04

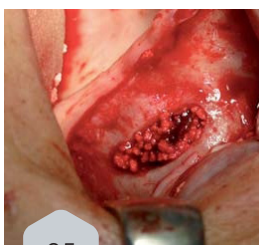

05

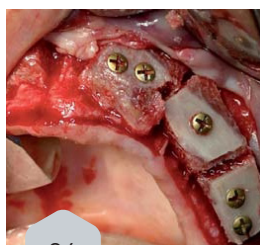

06

- › **01** Perforation of Schneiderian membrane
- › **02** Application of fibrin glue
- › **03** Closure of perforation
- › **04** Augmentation with **CERASORB® M**
- › **05** Filled sinus lift
- › **06** Lateral overlay osteoplasty

⁸Die Sinusbodenelevation und ihre Risiken – ZWP online – das Nachrichtenportal für die Dentalbranche (zwp-online.info)

Key takeaway from clinical performance

- › Easy to handle and safe to use
- › Promotes faster bone formation while maintaining structural integrity
- › Shows high resorption rates with simultaneous new bone deposition
- › Reduces surgical complications and enhances patient outcomes
- › New bone formation is seen between 4 – 6 months in the general population

Key takeaway of CERASORB® M advantage

- › **Biosynthetic, safe & effective:**
No biological risks, ensuring peace of mind for clinicians and patients.
- › **Optimized bone regeneration:**
Supports natural bone healing with predictable and reliable outcomes.
- › **Backed by scientific research:**
Proven clinical efficacy with extensive documentation and case studies.

Why choose CERASORB® M over Allografts & Xenografts?



CERASORB® M vs. Allografts

- › **No risk of disease transmission:** Allografts are derived from human donors and carry a potential risk of disease transmission, while **CERASORB® M** is fully biosynthetic and pathogen-free.
- › **Consistent quality:** Allografts vary in composition and quality based on donor source, whereas **CERASORB® M** offers a standardized, high-purity β -TCP formulation for reliable clinical outcomes.
- › **Faster and more predictable resorption:** Unlike allografts, which may take longer to integrate or be partially resorbed, **CERASORB® M** follows a controlled and complete resorption process in sync with natural bone formation within 4 – 6 months.

CERASORB® M vs. Xenografts

- › **Eliminates ethical & religious concerns:** Xenografts are derived from animal sources (bovine or porcine), which may raise ethical, religious, or immunological concerns. **CERASORB® M** is a fully biosynthetic alternative, suitable for all patients.
- › **No risk of immune reaction:** Xenografts may trigger foreign body responses, while **CERASORB® M** is biocompatible and seamlessly integrates into the patient's bone metabolism.
- › **Superior bone remodeling:** Unlike xenografts, which may remain as a scaffold for an extended period with incomplete resorption, **CERASORB® M** actively participates in bone remodeling and completely resorbs over time.




CERASORB® M Granules

Resorbable, pure-phase β -tricalcium phosphate

Ref No.	Size	Content	Packsize
9000 100 505	150 – 500 μm	0.5 cc	5
9000 200 505	500 – 1000 μm	0.5 cc	5
9000 201 005	500 – 1000 μm	1.0 cc	5
9000 300 505	1000 – 2000 μm	0.5 cc	5
9000 301 005	1000 – 2000 μm	1.0 cc	5
9000 302 005	1000 – 2000 μm	2.0 cc	5

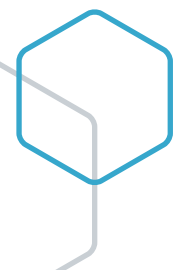

[illegible]



Head office

curasan AG

Lindigstraße 4
63801 Kleinostheim
Germany

P +49 6027 40900-0

F +49 6027 40900-29

M info@curasan.com

W www.curasan.com

Postal address

curasan AG

Ernst-Wiss-Straße 18
65933 Frankfurt am Main
Germany

curasan Inc.

1768 Heritage Center Drive,
Suite 204, Wake Forest, NC 27587
United States of America

P +1 919 941 9770

F +1 919 941 9775

www.curasaninc.com

Disclaimer

This document is intended exclusively for experts in the field, i.e. physicians in particular, and is expressly not for the information of laypersons. The information on the products and/or procedures contained in this document is of a general nature and does not represent medical advice or recommendations. Since this information does not constitute any diagnostic or therapeutic statement with regard to any individual medical case, individual examination and advising of the respective patient are, absolutely necessary and are not replaced by this document in whole or in part.

Distribution partner